

T330

# User Manual



Model: T330

# TABLE OF CONTENTS

---

The contents of the carton 01

---

Feature guide 02

---

Specification and main function 03

---

Key components 04

---

Manual /Auto Collapse Toggle  
Instructions 05

---

Instructions for unfolding  
the scooter manually 06

---

Manual folding and unfolding 07

---

Installation and adjustment  
of armrests 08

---

Basic trouble shooting 09

---

Scooter precaution and warranty  
information 10

---



Lithium battery




Remote control



Keys



Charger

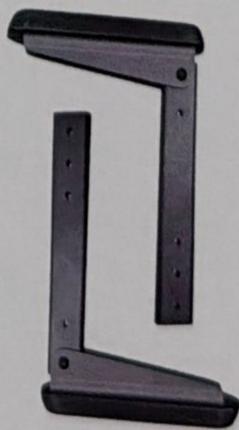

Armrests



Wrenches

# Specification

Folding size	400x450x700mm
Unfold size	980x450x950mm
Angle of climb	0-12°
Turning Radius	≤1.4 m
Max capacity	125 kg
Weight without battery	18.2kg
Maximum driving distance	15km+/23km+
Maximum speed	0-6 km/h
Motor	24V 200W
Lithium battery	24V 10Ah Or 24V 15Ah
Charging time	6-10H
Drive system	Rear Wheel Drive
Brake system	Electromagnetic
Front wheel diameter	PU 7" Solid Front Tire
Rear wheel diameter	PU 7" Solid Rear Tire
Color	Red and Blue



Please note that if you push the scooter too fast while the wheels are moving freely, the scooter will slow down.

1. Brake handle is on the left rear of the scooter. When the brake handle is in the lower gear, the scooter is in a no-brake state at this time, and the scooter cannot be started, but it can be implemented manually. It is necessary to push the brake handle upward to put it in the upper gear, at this time the scooter is in the normal state, and the scooter can be used normally.


Pull up is normal driving condition.




2. The rear axle is fitted with shock absorbers.



3. Removable lithium battery, small size, light weight, easy to carry.

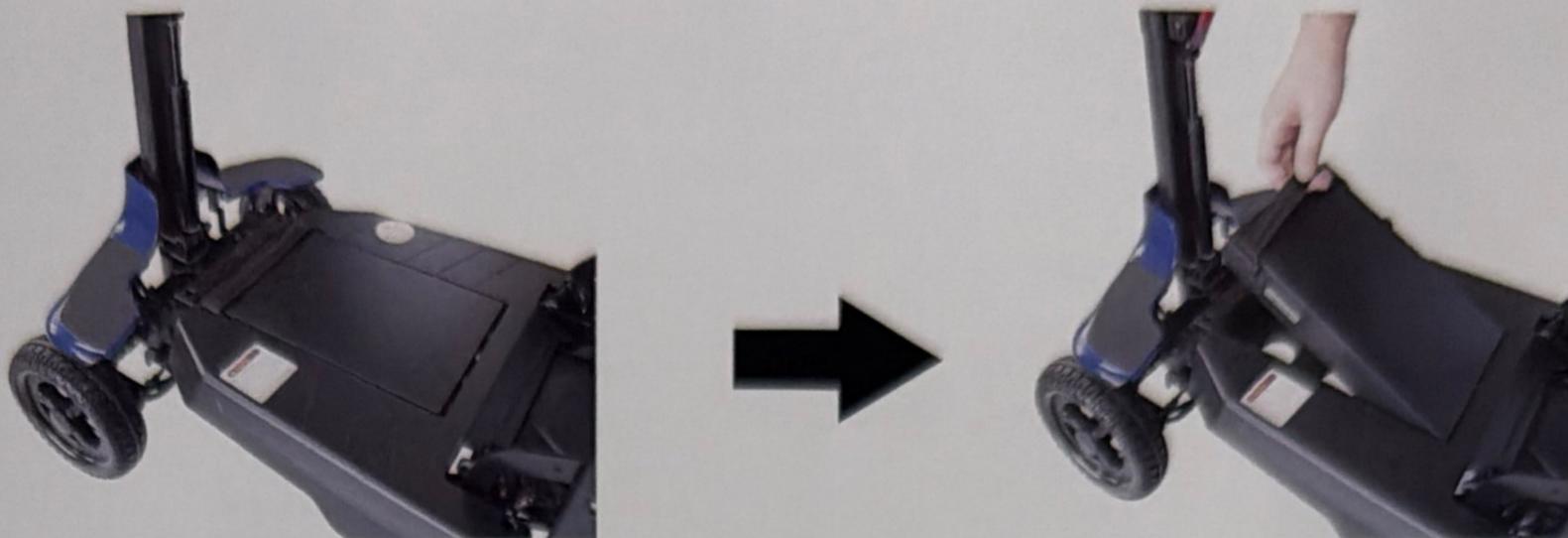

4. Front and rear PU solid tires.

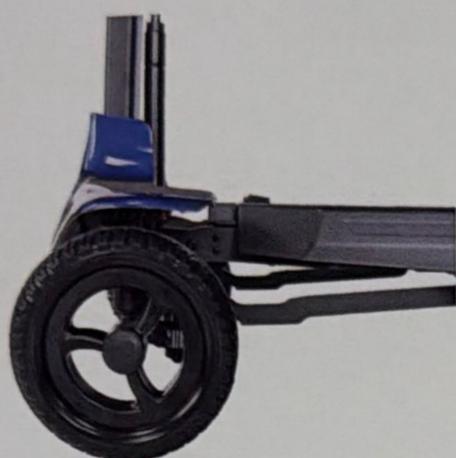


PU 7" Solid Front Tire




PU 7" Solid Rear Tire


5. Plug the charging cable into the charging port to charge the scooter.

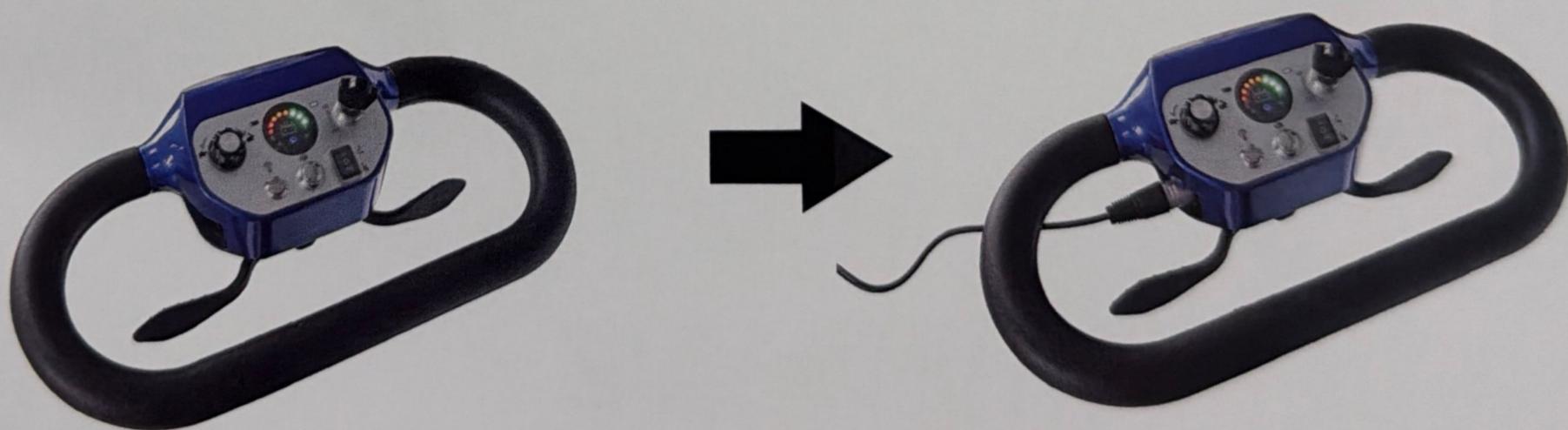

6.How to adjust the tiller

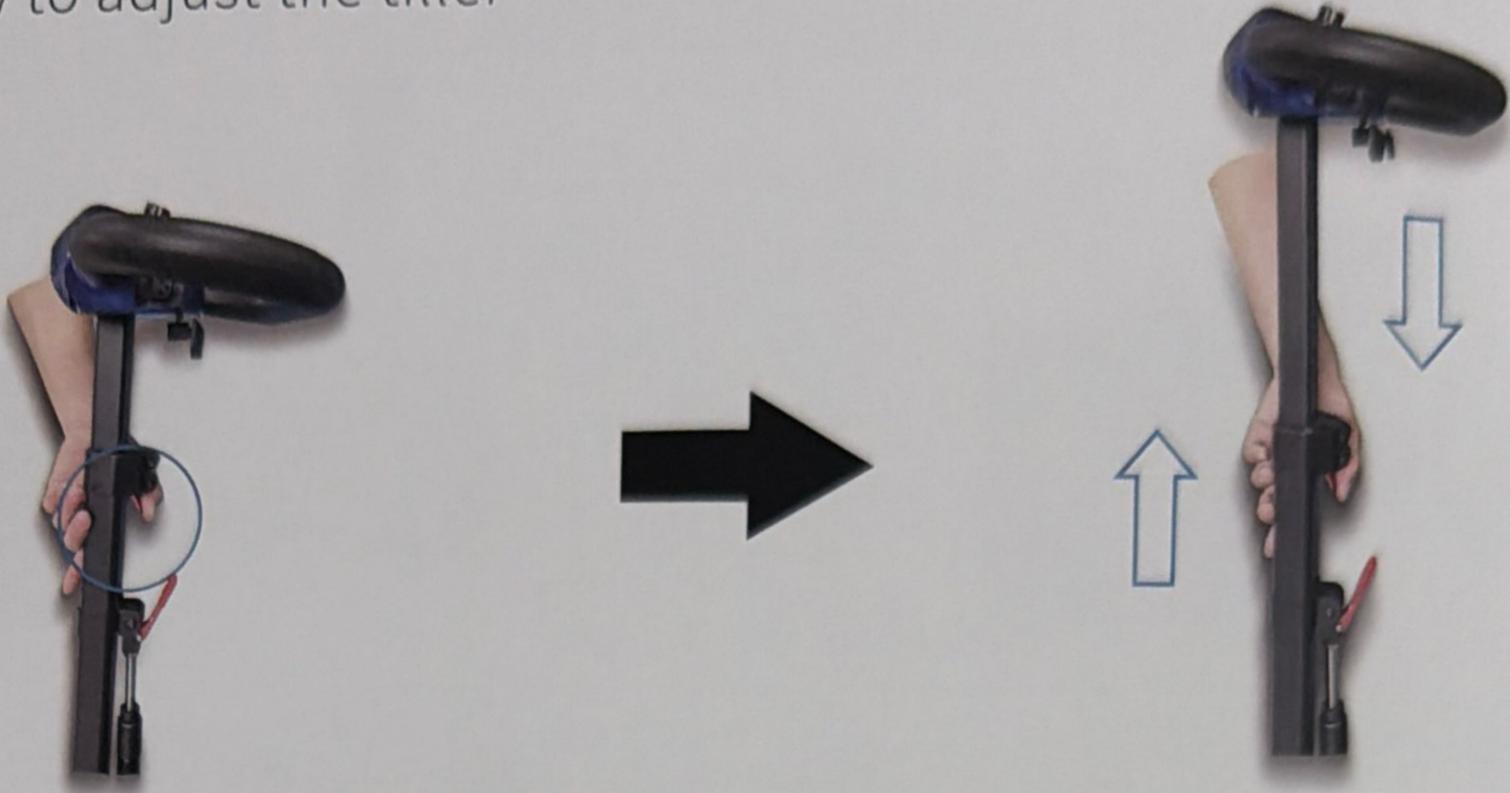

Height Adjustment Lock Lever

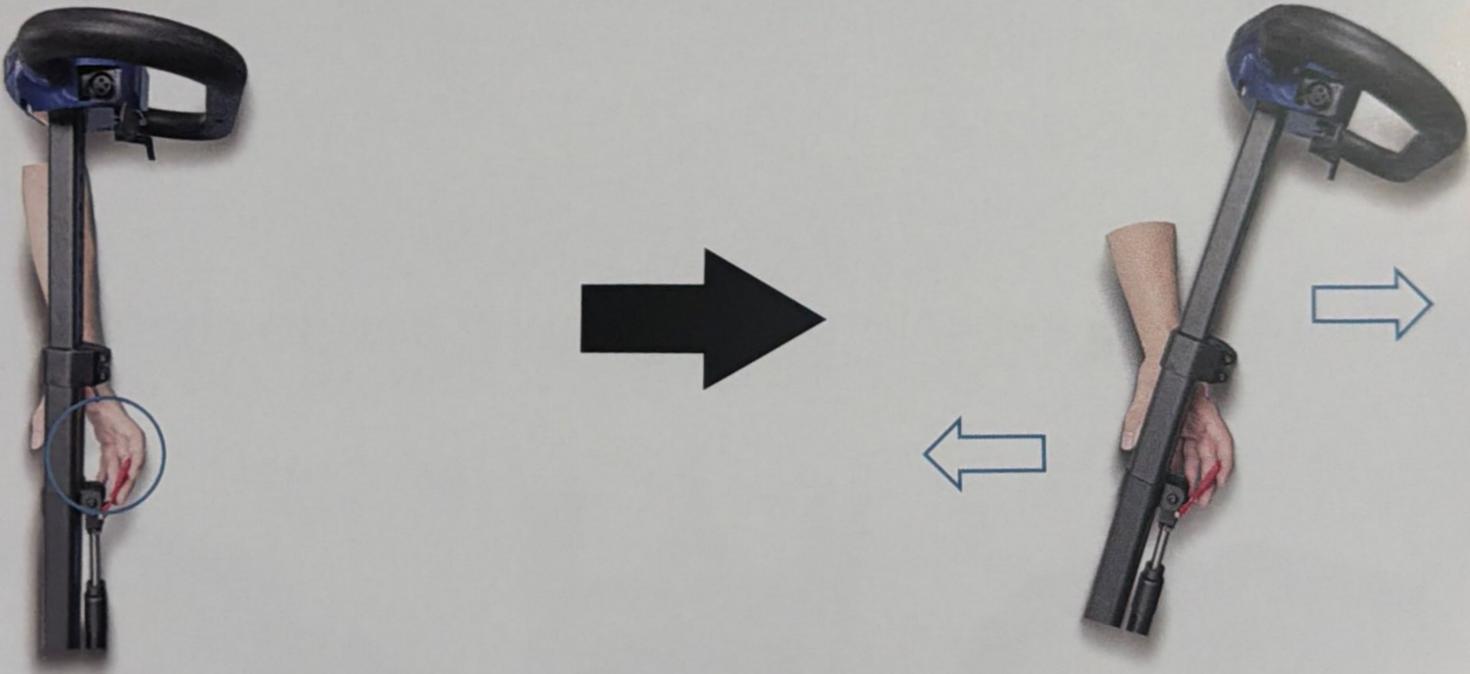


Tiller Tilt Adjuster

04


Key components

---

T330

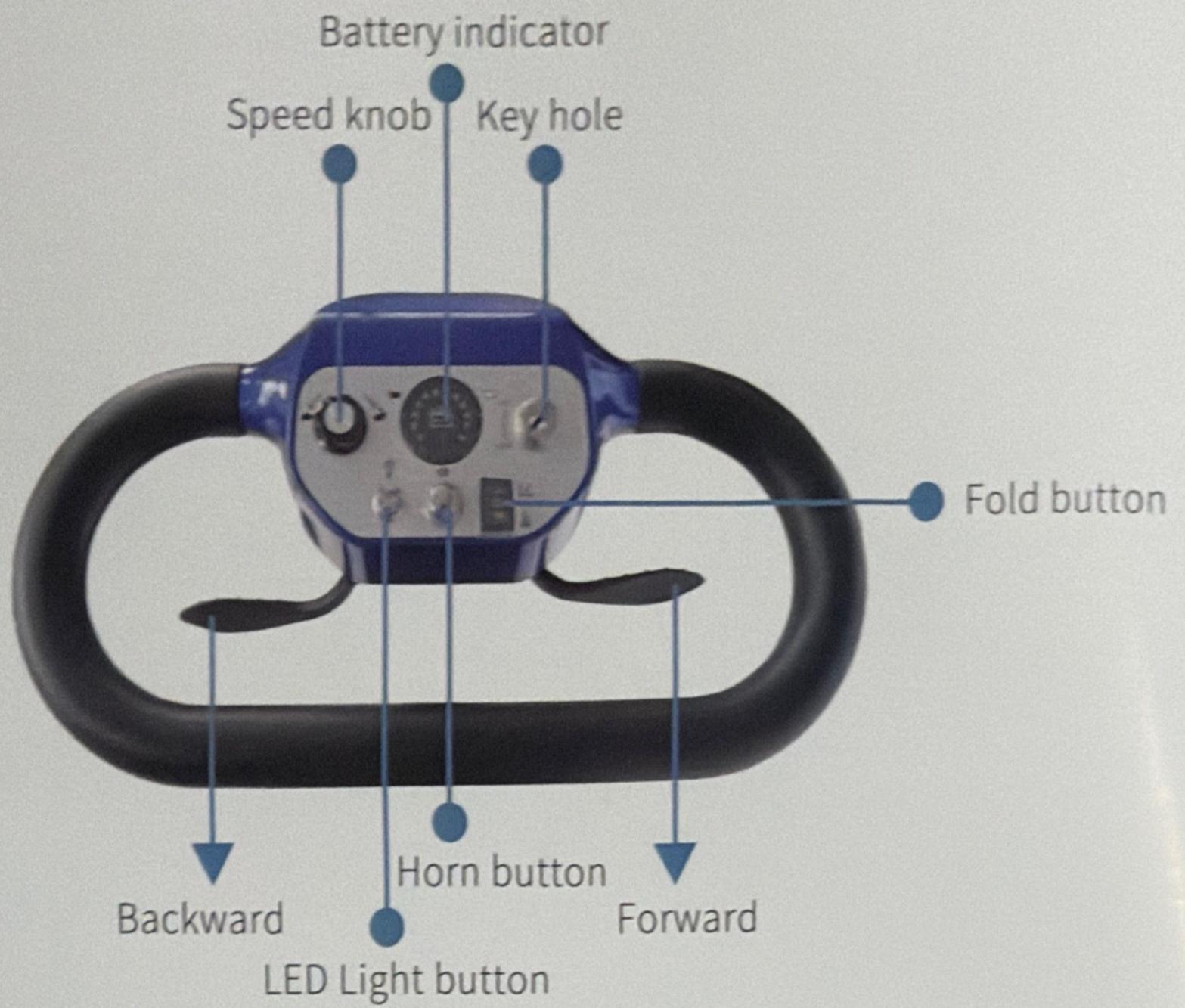

### Two charging methods

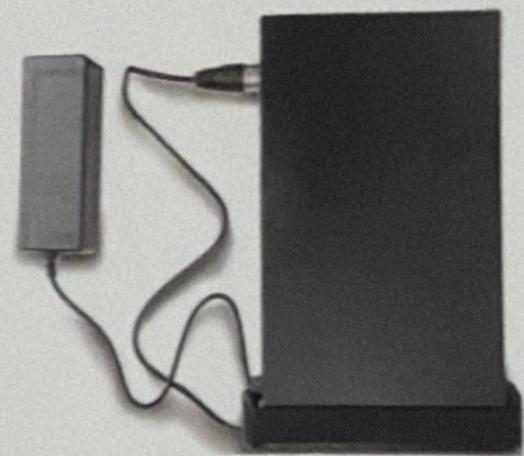


# 05


Manual /Auto Collapse  
Toggle Instructions

---

T330

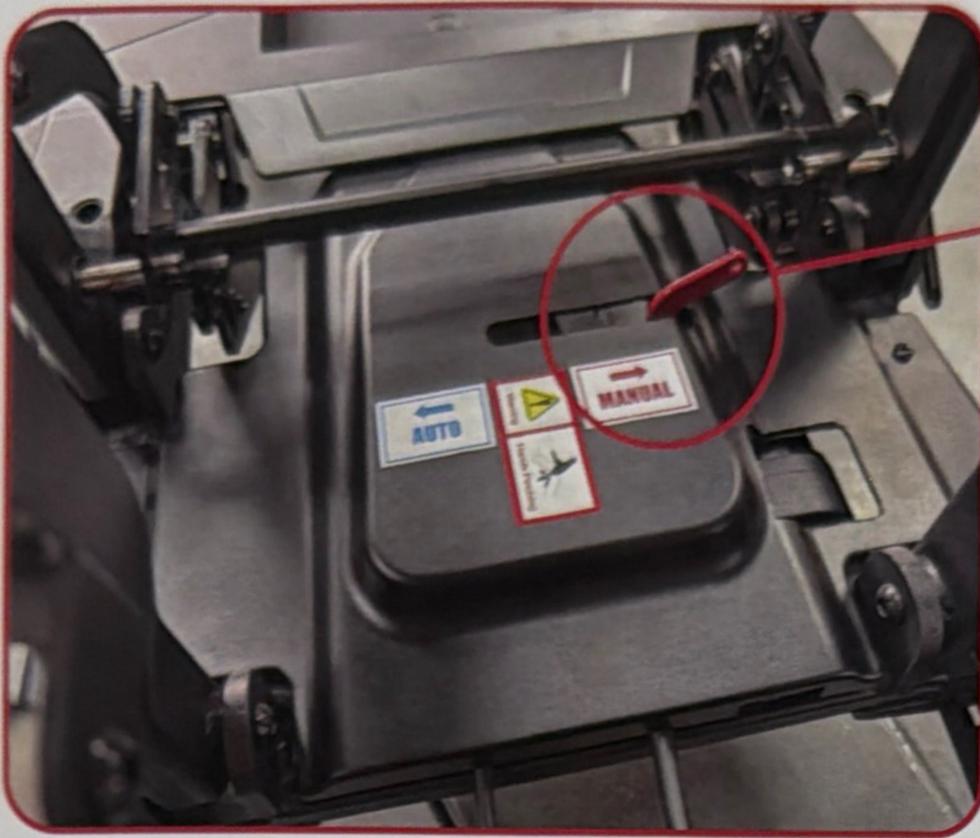

If the battery runs out or the Auto-fold function is fault, please push lever to Manual side in order to fold or unfold your scooter manually.

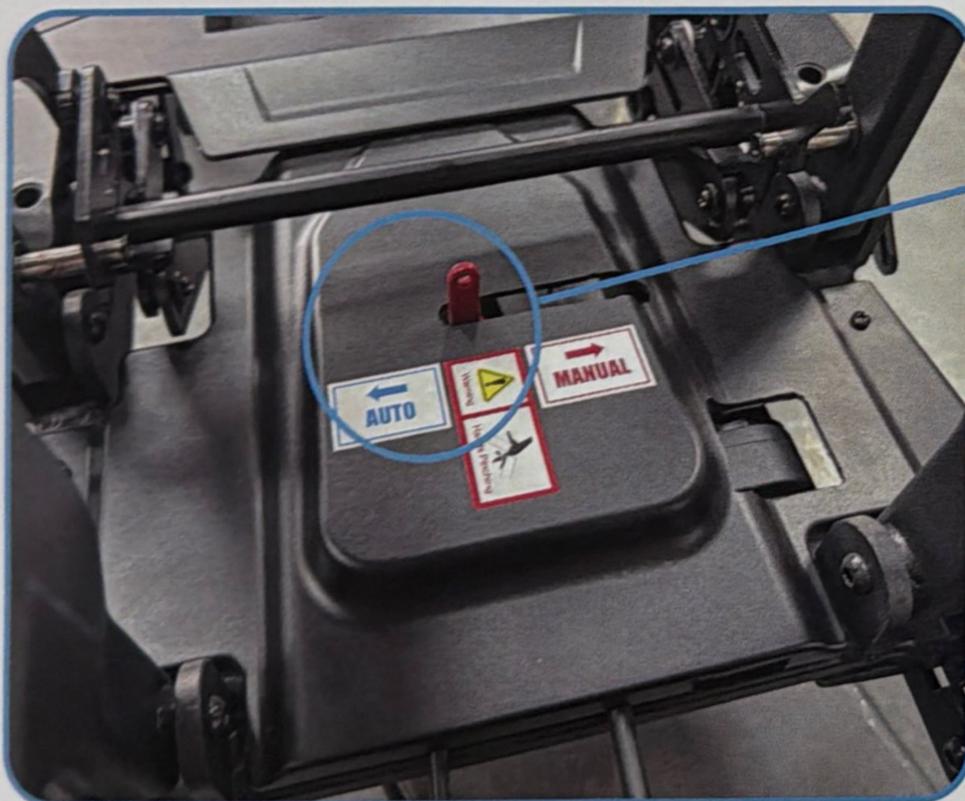

We recommend keep this lever on "Auto" position for scooter normal operation

# 06

Instructions for unfolding  
the scooter manually

---

T330

07

Manual folding and  
unfolding

---

T330

07

Manual folding and  
unfolding

---

T330



Press to unfold the scooter



Press to fold the scooter




### Unfolding

Please press and hold the button of UNFOLDING until the scooter is unfolded completely.



### Folding

Please press and hold the button of FOLDING until the scooter is fully folded.

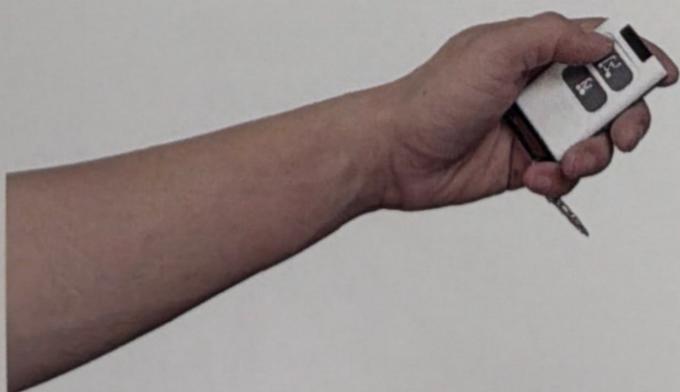

Long press the button of UNFOLDING to unfold the scooter.

08

Installation and  
adjustment of armrests

---

T330




01 Armrest default Angle.



02 Raise the armrests.



03 Adjust the height of the armrest screws to change their Angle.

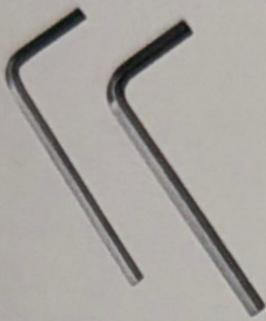

04 Use the included wrench to lock the screws.




05 Adjust the armrests to the proper Angle.



06 Lock the armrest bracket with the handle screws.

### Adjustable armrest width

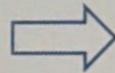

### Adjustable armrest height

